



Sinus Inhaler

USE AND CARE MANUAL



- Non medicated steam
- Adjustable steam output

IMPORTANT!
READ AND SAVE THESE INSTRUCTIONS

Model
VH200WAU

⚠ Important Safety Instructions

WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING: READ ALL INSTRUCTIONS BEFORE USING

⚠ DANGER To reduce the risk of electrocution:

1. **ALWAYS** turn off and unplug this product immediately after using.
2. **DO NOT** use while bathing.
3. **DO NOT** place or store product where it can fall or be pulled into a tub or sink. The appliance should always be placed on a firm, flat, waterproof and heat resistant surface. Be sure the appliance is in a stable position and the power cord is out of the way to prevent the appliance from being tipped over.
4. **DO NOT** place in or drop into water or other liquid.
5. **DO NOT** reach for a product that has fallen into water. Unplug immediately.

⚠ WARNING To reduce the risk of burns, electrocution, fire, or injury to persons:

1. The product should never be left unattended when plugged in.
2. Close supervision is necessary when this product is used by, on, or near invalids. **Keep out of reach of children.**
3. Use this product only for its intended use as described in this manual. **DO NOT** use attachments not recommended by the manufacturer.
4. **NEVER** operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged or dropped into water.
5. Keep the cord away from heated surfaces.
6. ⚠ **CAUTION: NEVER** move the appliance while in use. **It can spill hot water if tilted, shaken or tipped over causing injury or burns.**
7. **DO NOT** use appliance while sleepy or drowsy.
8. **DO NOT** use in area where aerosol products are being used or where oxygen is being administered.
9. **NEVER** block the air openings and vents of the product or place it on a soft surface, such as a bed or couch, where the air openings and vents may be blocked. Keep the air openings free of lint, hair, and other materials.

10. **NEVER** drop or insert any object into any opening or vent.
11. Unplug this product before filling. Fill (reservoir) with water only unless otherwise specified by manufacturer. **DO NOT** overfill. **DO NOT** fill with mask installed. Only pour water directly into Water Reservoir.
12. This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance. If the supply cord is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified persons in order to avoid a hazard.
13. ⚠ **CAUTION:** To avoid fire or shock hazard, plug the appliance directly into a 230V AC electrical outlet. **DO NOT** use an extension cord or a power strip.
14. The appliance should always be unplugged and emptied when not in operation or while being cleaned. Shut off and unplug the appliance before moving. Plug and unplug appliance with dry hands. **NEVER** pull by the cord to unplug.
15. ⚠ **CAUTION: DO NOT** move, lift, shake, tilt or disassemble while in operation or if it still contains hot water.
16. **DO NOT** operate the appliance without water.
17. **DO NOT** lift the appliance by the Mask. Lift only by the Base.
18. Appliance requires regular cleaning. Refer to and follow cleaning instructions.
19. **NEVER** add any essential oils, medications or inhalant liquids to the Water Reservoir.
20. **DO NOT** touch skin with exposed Vicks VapoPads or place on furniture, fabrics, bedding or plastics.
21. **DO NOT** operate outdoors. This appliance is intended for indoor use.

22. **DO NOT** attempt to repair or adjust any electrical or mechanical functions on this appliance. Doing so will void your warranty.
23. For residential use only.
24. When the appliance is used in a bathroom, unplug it after use since the proximity of water presents a hazard even when the appliance is switched off.
25. For additional protection, the installation

of a residual current device (RCD) having a rated residual operating current not exceeding 30 mA is advisable in the electrical circuit supplying the bathroom. Ask your installer for advice.

26. After use the appliance should be cleaned to avoid the accumulation of grease and other residues.

SAVE THESE INSTRUCTIONS

Accessories


Vicks VapoPads

Vicks VapoPads are designed to work with your Steam Inhaler to provide soothing vapors. The Steam Inhaler accepts up to two Vicks VapoPads. Enjoy up to 8 hours of soothing comfort per pad. (Two VapoPads Included)








Vicks Sinus Inhaler

1. Mask
2. VapoPad Tray
3. Top Housing
4. Steam Disk Controls
5. Base
6. Water Reservoir
7. Power Switch
8. Vents
9. Heating Element

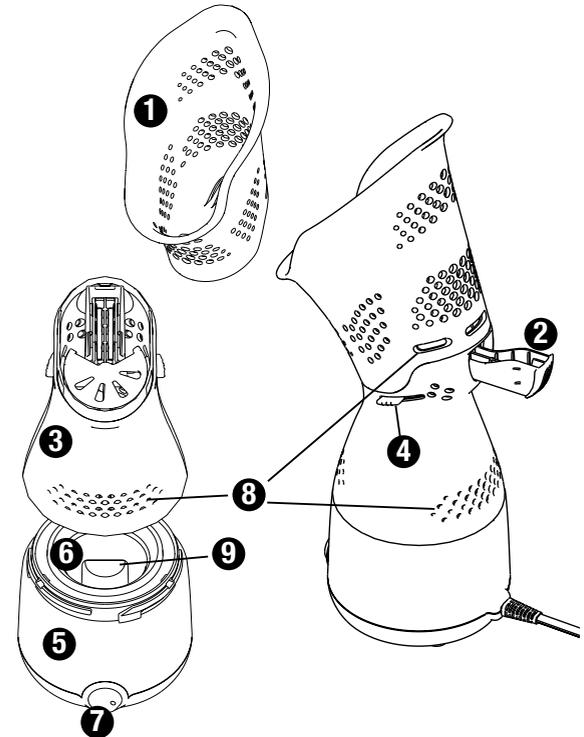



Using Your Personal Steam Inhaler

1. Fully unravel cord. Failure to do so could cause overheating and possible fire hazard. **DO NOT** use with extension cords or power strip. Place Inhaler on a flat, stable surface for use.
 2. Unlock the Top Housing from the Base by twisting the Top Housing counter clockwise. When fully unlocked, the arrow ▼ on the Top Housing will align with the unlock 🗝️ symbol on the Base. Lift the Top Housing straight up and set aside.
 3. Fill Water Reservoir with up to 2.5 tablespoons (40 mL) of cool tap water. **DO NOT** fill above Heating Element. Use with water only. **DO NOT** add any essential oils, medication or inhalant liquids to the Water Reservoir. These will damage the unit and void the warranty.
 4. Place Top Housing on the Base and twist clockwise to lock. When fully locked, you will feel a click and the arrow ▼ on the Top Housing will align with the lock 🔒 symbol on Base.
 5. Place Mask on Top Housing and gently push down to ensure Mask is secure.
- ⚠️ **CAUTION: NEVER** move, lift, shake or tilt the Inhaler while it is in use or while it contains hot water. **DO NOT** remove the Mask or Top Housing after turning ON the Inhaler. Steam is produced by boiling water; tilting or shaking unit can cause water to spill and possible burns.
- ⚠️ **CAUTION: DO NOT** block any vents.
6. Connect cord to electrical outlet. **DO NOT** use with extension cord or power strip. Press the On/Off Switch to the ON position. The Indicator Light will illuminate.
 7. The Sinus Inhaler will take approximately 6 minutes to begin making steam. Use caution, the Water Reservoir and water will become hot.
 8. Place your face over the Mask. **DO NOT** rest your face on the Mask. Breathe normally through your nose and mouth, inhaling steam into your nasal and throat passages. Continue application for five to fifteen minutes.

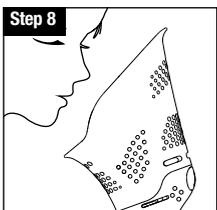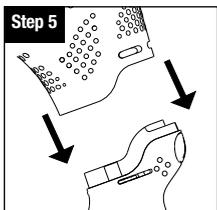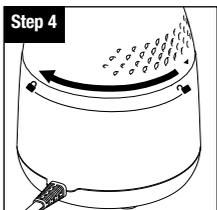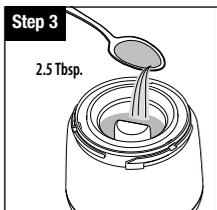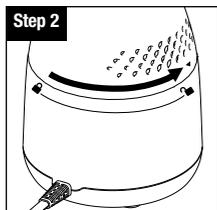

9. Adjusting steam intensity:

- A. For More Steam:** Move the Steam Disk Control clockwise for maximum steam.
- B. For Less Steam:** Move the Steam Disk Control counterclockwise for minimum steam. If at any time the application becomes too intense, stop application immediately.

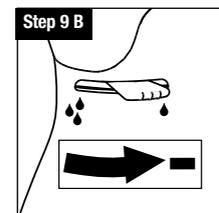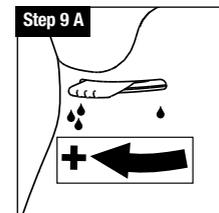

10. When application is complete, move Power Switch to the OFF position. The Indicator Light will go out. Disconnect plug from electrical outlet. Allow the appliance to cool for at least 20 minutes before moving or refilling it (see Care and Cleaning section). **DO NOT** operate Sinus Inhaler when empty.

NOTE: Sinus Inhaler will automatically shut-off when water runs out. If this happens, move Power Switch to OFF position and unplug unit.

Using Vicks VapoPads

Your Vicks Sinus Inhaler can be used with Vicks VapoPads, to provide up to 8 hours of soothing vapors. Two samples of the VapoPads are included with your device.




To use your unit with scent pads, ensure that the unit is OFF and follow these instructions:

1. Gently slide out the VapoPad Tray from the back of the Top Housing. Note: Only slide Tray out. **DO NOT** attempt to detach the Tray from the Top Housing.
2. Open VapoPad sachet by tearing notch on pad bag. **DO NOT** touch pad with hands. If pad is touched, **DO NOT** rub face or eyes as it may cause irritation.
3. Insert Vicks VapoPad with angled end at the front of the tray. Up to two scent pads can be inserted, one in each slot of the VapoPad Tray. Push VapoPad Tray in to close.
4. When the VapoPad is dried and no longer providing scent, discard.

NOTE: DO NOT operate Sinus Inhaler if VapoPad Tray is missing or not secured.

For use with Vicks VapoPads only. Not for use with essential oils or inhalants. These will damage the unit and void the warranty.

This Sinus Inhaler (with or without Vicks VapoPads) is not recommended for use by children under 8 years.

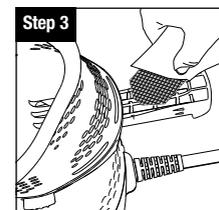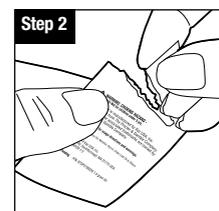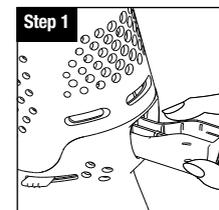

Using Vicks VapoPads (continued)

VapoPads Precautions:

- Not for use by children under 8 years.

Scent Pad Ingredients: VSP-19 Series: Eucalyptus Oil, Menthol, Glycol, Cedar Leaf Oil & Others.

Care and Cleaning

⚠ CAUTION: ALWAYS disconnect the plug from the electrical outlet and let the Sinus Inhaler cool down for at least 20 minutes before cleaning.

After Each Use

1. Remove Mask and Top Housing and wipe dry with a clean cloth or paper towel.
2. If any water is left in the Water Reservoir, pour water out and wipe Reservoir and Base with a dry, clean cloth or paper towel. **NEVER** immerse the Base in water or any other liquid.
3. Replace Top Housing and Mask; store in a cool, dry place.

Weekly Maintenance

1. Remove Mask and Top Housing.
2. Pour 2.5 tablespoons (40mL) of undiluted distilled white vinegar into the Water Reservoir and let sit for 10 minutes.
3. Empty the distilled white vinegar and rinse out Water Reservoir with fresh, clean tap water. Use caution to not let water get into the electrical areas.
4. Dry the Water Reservoir and Base with a clean cloth or paper towel.
5. Wash Mask with mild soap and warm water then carefully rinse off all soap with fresh, clean tap water. Dry with a clean cloth or paper towel.
6. Replace Top Housing and Mask. Store in a cool, dry place.
7. To clean the VapoPad Tray, wet the tip of a cotton swab and wipe slots.

NOTE: The Top Housing is dishwasher safe. It may be cleaned in the Top Rack of the dishwasher or by hand as noted in the instructions. **DO NOT** place in the bottom rack of dishwasher. **DO NOT** attempt to disassemble the Top Housing. The Base should **NEVER** be put in the dishwasher.

Care and Cleaning (continued)

After Long Periods of Non-Use

If you haven't used your Sinus Inhaler for two weeks or longer, follow the Weekly Maintenance Instructions before the next use.

Condition of Sale

As a condition of sale, the purchaser assumes responsibility for the proper care and use of this appliance in accordance with our printed instructions. The purchaser or user must judge for himself or herself when to use it and length of use.

Specifications

Electrical rating: 220-240V~ 50-60Hz 70W

Warranty

This product has been produced with the greatest care for normal, household use. Following directions carefully will ensure dependable operation. If for any reason (other than misuse) you are dissatisfied with your Vicks Sinus Inhaler, we will repair or replace it, at our option, at no charge during the 2 year warranty period. Modification to the product by the consumer is not authorised and voids the warranty.

If you experience unsatisfactory operation, please contact Felton Grimwade & Bosisto's. If repair is required under the warranty period, the product should be returned with proof of purchase to Felton Grimwade & Bosisto's service centre.

The purchaser assumes responsibility for the proper care and use of the product in accordance with the printed operating manual. The purchaser or user must make his or her judgment as to when to use the product and the length of use. Keep the operation manual available as it is your guide to safe, efficient operation.

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.

For warranty claims please visit www.fgb.com.au/faulty-vaporiser

Distributed by:
Felton Grimwade & Bosisto's Pty. Ltd.
61-81 Clarinda Road (Po Box 74)
Oakleigh South, Victoria 3167, Australia



Free call: 1800 655 841
Website: www.fgb.com.au

© 2023, All rights reserved.



Helen of Troy Macao Limited
Avenida Xian Xing Hai No.105, Centro Golden Dragon,
14 andar H,I,J,K, Macau

Certain trademarks used under license from The
Procter & Gamble Company or its affiliates.

Made in China.

P/N: A008713R0
Date: 19JUN23

Helen of Troy		Creative Department Artwork Specifications				
VICKS Steam Inhaler						
VH200WAU						
Artwork Part #: A008713R0 Dieline Part #: NA						
Subject: Owner's Manual Region: AP-AU						
Flat Size: MM: W240 x H200 IN: W9.4488 x H7.875 Folded Size: MM: W120 x H200 IN: W4.724 x H7.875						
Scale: 1/1						
Material: 70g wood paper						
Page Count: 8						
Revision: 5 Date: 19JUN23 Release Date: Re-Release Date:						
Colors		Spot Colors				
Cyan 0%	Magenta 0%	Yellow 0%	Black (K) 100%	PMS 0000	PMS 0000	PMS 0000
Dielines do not print						
Coating: NA						
Bindery: Saddle Stitch						
Special Instructions:						
Quality Requirement of Artwork Clarification Process of Artwork Printing: Meet Eng-QS-06&02						
Helen of Troy Creative Services Marlborough, MA 01752 USA Questions? Contact your representative						